



I-Ring® Pupil Expander Usage Guide

Introduce the I-Ring into the Anterior Chamber

1
Introduce the Inserter through the primary incision after injecting viscoelastic into the Anterior Chamber. Position the tip centrally over the lens and introduce the I-Ring.

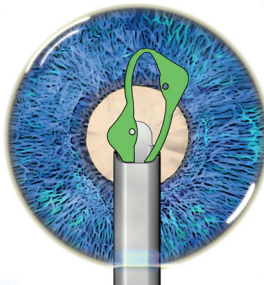

2
Completely retract the prongs (a) before removing the Inserter (b) from the primary incision.

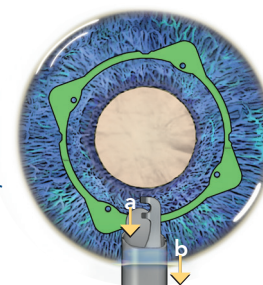


Engage the I-Ring to the Iris


1
Use a Sinsky Hook to secure the distal channel to the iris.



2
Next secure the proximal channel to the iris.

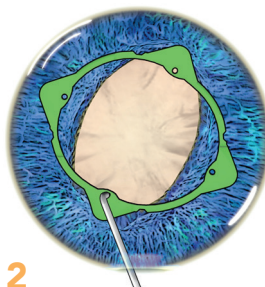


3
Continue to secure the two lateral channels to the iris.

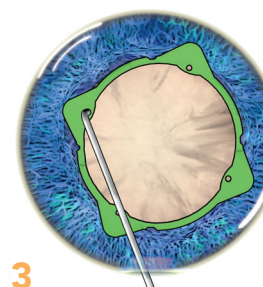


Remove the I-Ring from the eye


1
Use a Sinsky Hook to disengage the distal channel and place it onto anterior iris.

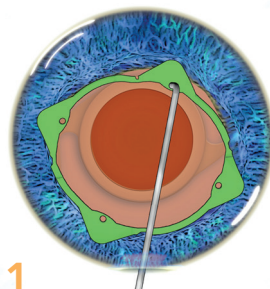


2
Next disengage the proximal channel and place it onto anterior iris. Proceed to disengage the two lateral channels.

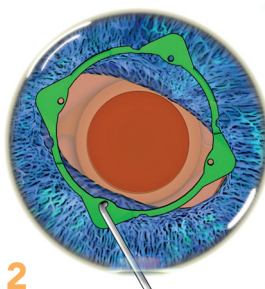


3
With the Inserter in the Anterior Chamber, extend the prongs over the I-Ring before positioning the cannula

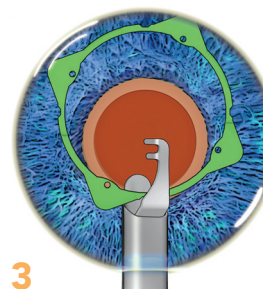

4
Capture the hinge between the prongs of the Inserter.

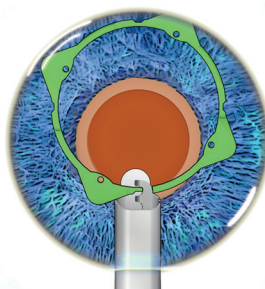

5
Completely draw the I-Ring into the Inserter before withdrawing from the primary

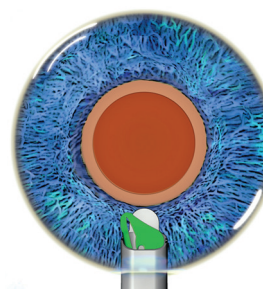

Key:

